## Gebrauchsanweisung els unbond

### 1. Produktbeschreibung

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## Instruction for use els unbond

### 1. Product description

Light-curing, 1-component self-etching adhesive to create a permanent marginal-gap-free adhesion between the tooth structure (dentine, enamel) and the light-curing filling/ fixing material.

### 2. Composition

ethanol, water, BisEMA, methacrylated phosphoric salt, silica, polyalkenoate methacrylized, initiators

### 3. Intended Use

SareMCO dental bonding agents promote adhesion of direct or indirect prosthesis for reconstruction or correction of functionally compromised natural dentition (e.g., deficient teeth).

### 4. Indication

1. Direct restorations with light-curing composite-based materials.  
2. Indirect restorations: at the use of light-curing composite cements to fix inlays, onlays, crowns and bridges.

### 5. Contra-indication

Opened pulp, pulpitis, known allergy to methacrylates.

### 6. Patient target group

els unbond is suitable for use in all patients without any age or gender restrictions.

### 7. User

els unbond should only be used by a professionally trained dental practitioner.

### 8. Side effects

In individual cases, contact allergies have been described to products of a similar composition. To avoid pulp reactions, it is recommended to cover the exposed dentine in the pulp area with suitable underfilling materials (preferably with calcium hydroxide). **Note:** *els unbond* does not contain HEMA, TEGDMA or BisGMA.

### 9. Interactions

Avoid underfilling materials which may hinder polymerization owing to their ingredients. All phenolic compounds, such as zinc oxide eugenol or preparations containing thymol, belong to this category.

### 10. Processing stages

#### 10.1. Drying

Due to the following application of the filling material, a drainage of the working field is required. Avoid any contamination of the cavity with blood or saliva. A rubber dam is recommended.

#### 10.2. Cavity preparation

Prepare the cavity as usual. Undercuts and beveled margins are recommended to improve adhesion and the margin fit. Clean and dry the cavity. Avoid overdrilling.

#### 10.2.1. Etching of tooth substance (optional)

By applying *els unbond* clinically sufficient bond strength values are achieved. By selective enamel etching or in total-etch technique even higher adhesion values can be achieved.

#### 10.2.1.1. Total-etch technique

Apply the etching gel and condition in correspondence to instruction for use of the product. SAREMCO recommends *cmf etch*.

#### 10.2.1.2. Selective enamel etching

Apply the etching gel condition in correspondence to instruction for use of the product. SAREMCO recommends *cmf etch*. Sufficiently etched enamel appears matt and white.

**Note:** Possibly use the total-etch-technique in cavities limited by dentine. In this case the dentine should not be etched for more than 15 seconds and only dried briefly after rinsing.

#### 10.2.2. Non-Etch

Do not overdry the dentine. A slightly moist cavity surface is optimal. Without preparation (e.g., cervical defect), thoroughly clean the surface and roughen the enamel margin with a finishing diamond.

#### 10.3. Application

Apply *els unbond* to enamel/dentine in a not too thin layer with a brush and massage into the tooth substance for 20 seconds. **Note:** *els unbond* is light-curing, so avoid too intense ambient light. Dim the surgical light during the application.

#### 10.4. Drying

Dry the adhesive layer with an air blower for at least 5 seconds.

#### 10.5. Polymerizing

Polymerize the bonding layer for 20 seconds with blue light (halogen or LED light). When using dual-curing luting composites, light-curing of the luting composite is mandatory.

**Note:** Do not use *els unbond* for opaque restorations or deep cavities where complete light-curing is not guaranteed.

#### 10.6. Notes for fixing Maryland bridges

When fixing Maryland bridges with *els unbond*, the enamel must be roughened or etched with phosphoric acid gel (e.g. SAREMCO *cmf etch*) prior to use of the bond. Self-etching bonding agents are less effective on non-prepared enamel. Excessive amount of bond material onto unpolished and unbeveled enamel can cause edge staining. For optimal bonding results with *els unbond*, the enamel should be prepared in advance in class III, IV, V and for diastema closures.

### 11. Storage

Close *els unbond* bottle tightly after use. Store in an upright position to ensure the back-flow of the liquid. Do not expose light-curing products to direct sunlight or operating light. *els unbond* was developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store at temperatures between 4°C - 28°C / 39°F - 82°F. If room temperature exceeds 28°C / 82°F storing in the refrigerator is recommended. Do not freeze! Constant temperatures above 28°C/ 82°F can reduce the shelf-life of the product.

### 12. Batch number and expiry date

The batch number should be specified to identify products in the case of enquiries. Products should no longer be used once the expiry date has elapsed.

### 13. Precautionary measures

Close containers after each use with the right lid. Keep out of reach of children. For dental use only. Commercially available medical gloves do not provide protection against the sensitization effect of methacrylates. If the product comes into contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice.

### 14. Emergency measures

In case of direct contact with the oral mucosa, rinse with water. In case of contact with the eyes, rinse thoroughly with water. Consult an eye specialist.

### 15. Hygiene

Use application instruments for one patient only. Dispose products away from patients to avoid contamination.

### 16. Warranty

The warranty was developed for use in dentistry and must be processed in accordance with the instructions for use. For further damages, namely that caused by non-compliance with the instructions for use or other improper handling or inappropriate use of a product, any liability is rejected. Our liability is restricted to the quality of our products. In the case of a product being of defective quality, only its value is replaced. It is the responsibility of the user to check, before using the products, whether they are suitable for the intended purpose. He expressly assumes all risks associated with using the product and is solely responsible for any resulting damages. Safety data sheets and technical data sheets are available on the website of SAREMCO Dental AG.

### 17. Other notes for Europe

If the user and/or patient become aware of serious incidents connected with application of the product, they are to be reported to the manufacturer and the responsible authorities of the state in which the user and/or patient resides. The Summary of Safety and Clinical Performance of SAREMCO Products can be found in the European database on medical devices (EUDAMED-<https://ec.europa.eu/tools/eudamed>).

### 18. Production / distribution

SAREMCO Dental AG  
Gewerbestrasse 4  
CH-9445 Rebstein / Switzerland  
Tel: +41 (0) 71 775 80 90  
Fax: +41 (0) 71 775 80 99  
info@saremc.ch  
www.saremc.ch

Edited 10-2022 | D600169

Class III medical devices  
Class III medical devices (Canada)

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